



# CryoPen<sup>®</sup> XP

## Veterinary version



- Ø 1-3mm  
Power: ■
- Ø 2-4mm  
Power: ■■
- Ø 3-5mm  
Power: ■■■
- Ø 5-8mm  
Power: ■■■■

With better control of procedures the CryoPen XP sets new standards for superb outcomes.

The CryoPen XP dispenses a fine jet of nitrous oxide under high pressure which allows you to work with millimeter precision.

It offers the ideal solution for accurate, quick, effective, stress-free treatment of all skin imperfections.

### Instrument

This instrument is well prepared for all treatment options. With the possibility to use both 8g and 16g cartridges and the complete range of applicators, CryoPen XP gives additional value in all situations.

### Cartridges

8g N<sub>2</sub>O cartridges: 1 box containing 24 cartridges  
1 carton containing 12 boxes

16g N<sub>2</sub>O cartridges: 1 box containing 6 cartridges  
1 carton containing 12 boxes

Once punctured the pressure will remain without loss for many weeks. Shelf life 36 months. Storage at room temperature.

### CryoPen XP set with 4 applicators

Blue ring applicator: Ø 1-3mm  
flow time 200-230sec (8g) / 370-430sec (16g)

Grey ring applicator: Ø 2-4mm  
flow time 145-175sec (8g) / 270-340sec (16g)

Green ring applicator: Ø 3-5mm  
flow time 95-110sec (8g) / 175-215sec (16g)

Yellow ring applicator: Ø 5-8mm  
flow time 60-70 (8g) / 110-130sec (16g)

### What kind of lesions?

Benign skin lesions: Adenoma, Distichiasis, Epulis, Eyelid adenoma, Meibomian gland dysfunction, Sarcoid, Warts, and more....

### About the procedure

No special preparation is required. Treatment is applied for 2 to 25 seconds on each lesion. In larger areas circular or crossing movement is required to include all tissue.